Newsletter

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
The screenshot shows the 'UK NEQAS Haematology' software interface. At the top left is the UK NEQAS logo and 'Haematology and Transfusion'. The main title is 'UK NEQAS Haematology' with 'version: 2.06' on the right. Below this is a 'Sample Entry Details' section with buttons for 'Back to List', 'Save', and 'Submit', and a link for 'Blank data entry form'. The main heading is 'BLOOD FILMS Manual Differential Leucocyte Count'. Below this, there are fields for 'Distribution Number: 1702DF', 'Participant:', 'Issued: 05/06/2017', and 'Closing: 15/06/2017'. At the bottom, there are input fields for 'Received:' and 'Analysed:'.

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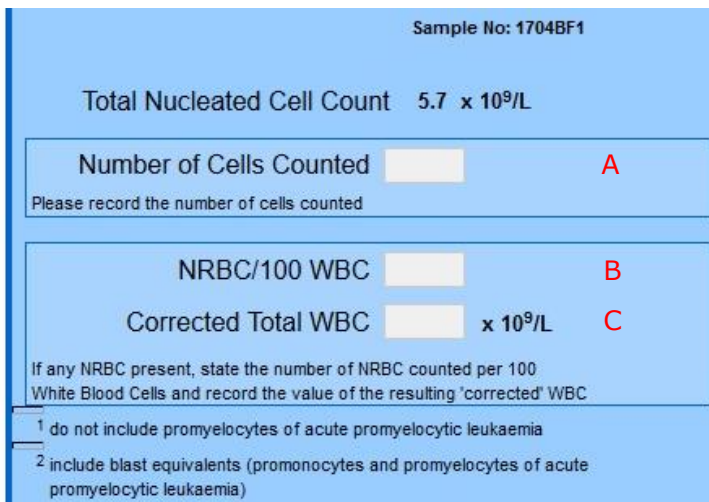The screenshot shows a data entry form for 'Sample No: 1704BF1'. It displays 'Total Nucleated Cell Count 5.7 x 10⁹/L'. There are three main input sections: 1) 'Number of Cells Counted' with a text box and a red 'A' label, with the instruction 'Please record the number of cells counted'. 2) 'NRBC/100 WBC' with a text box and a red 'B' label. 3) 'Corrected Total WBC' with a text box, 'x 10⁹/L', and a red 'C' label. Below these are instructions: 'If any NRBC present, state the number of NRBC counted per 100 White Blood Cells and record the value of the resulting 'corrected' WBC'. There are also two footnotes: '1 do not include promyelocytes of acute promyelocytic leukaemia' and '2 include blast equivalents (promonocytes and promyelocytes of acute promyelocytic leukaemia)'.

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Total Nucleated Cell Count: 10.0 x 10⁹/L
NRBC/100 WBC: 25

$$\frac{10.0 \times 100}{25 + 100} = 8.0 \times 10^9/\text{L} \quad \text{Conteggio corretto di leucociti (Corrected Total WBC)}$$

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Cell Type	Percentage	Absolute x 10 ⁹ /L
Neutrophils (inc Bands)	%	
Lymphocytes	%	
Monocytes	%	
Eosinophils	%	
Basophils	%	
Metamyelocytes	%	
Myelocytes	%	
Promyelocytes ¹	%	
Blasts ²	%	
Total:	%	

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